




Lipedema
Efficient
pain relief

DEEP OSCILLATION® Personal Home

Treatment at home or on the move

   **PHYSIOMED®**
technology for therapy

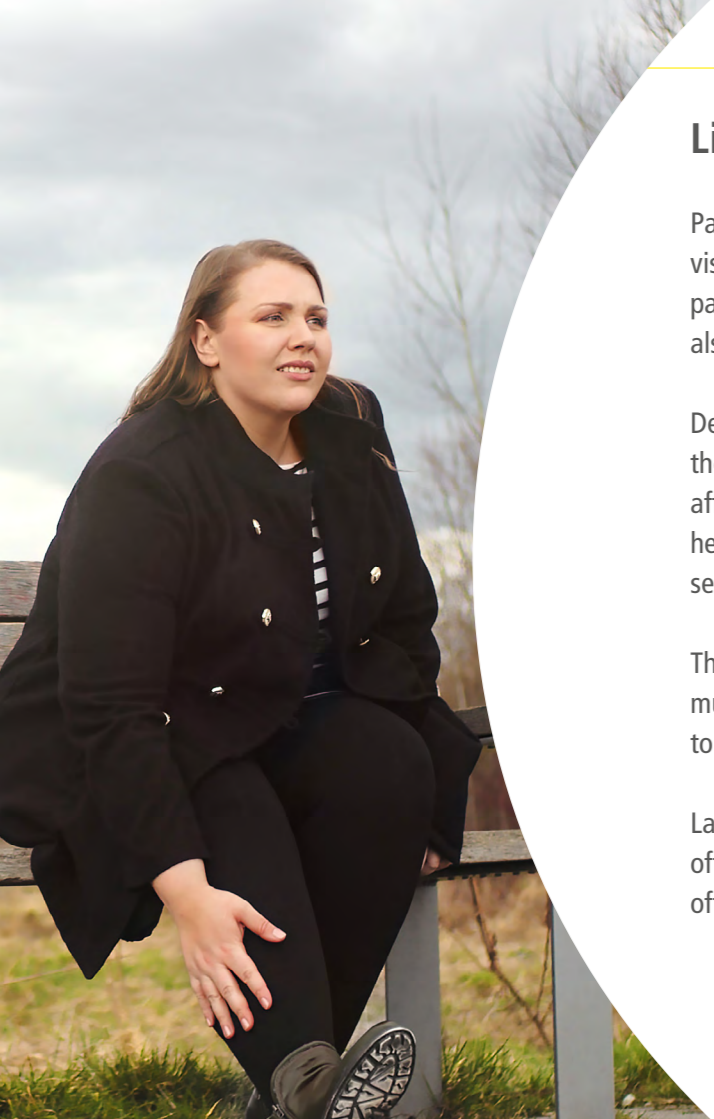

Lipedema – how those affected suffer

Patients with lipodystrophy not only have to struggle with their visual appearance. They also have to deal with physical symptoms in particular. There is no prospect of healing and options for treatment are also limited.

Depending on the stage, the thighs or the entire lower limbs from the hips to the ankles will be heavily swollen. The parts of the body affected feel heavy and tight, and are sensitive to pressure and prone to hematomas. Even light touches are often experienced as an unpleasant sensation by patients.

The disease is progressive. In later stages a secondary lymphedema must be reckoned with, in which hands and feet may also swell up due to inadequate transport of the lymph and fluid retention in the tissue.

Last but not least is the mental burden on sick women. Diagnosis is often late. Without further diagnosis, sufferers are often glibly written off as “too fat”.

DEEP OSCILLATION® Personal Home – relief at all stages and in all situations

Stage 1

smooth skin surface, thickened subcutaneous fatty tissue, fine nodular fat structure, “saddlebags” on the thighs

Stage 2

uneven skin surface, coarse nodular fat structure, increasing disproportionation of trunk and legs/arms

Stage 3

Induration (hardening) of tissue, sharp increase in leg volume, formation of pockets in the skin and of knock-knees (lipedema stage classification)

Lipo-lymphedema

additional deposit of lymph in hands and feet

Efficient deep treatment for home use – DEEP OSCILLATION® Personal Home

Deep oscillation has been used successfully for years by lymph therapists. It intensifies the effect of MLD (Manual Lymphatic Drainage) and provides relief for patients with lipedema and lymphedema.

“I detected incredible improvements. [...] From the moment we started using Deep Oscillation, pressure sensitivity disappeared and even the tendency to black and blue marks has been reduced.” Lisa W., Neumarkt

For the first time, PHYSIOMED presents this instrument for **home use**. With this, patients have the chance to treat themselves quickly and easily at any time.



The effects of deep oscillation are clinically proven:

- pain reduction
- anti-inflammatory effect
- edema resorption
- promotes healing of wounds
- actively works against the formation of fibroses
- improves tissue supply
- reduction in erythemas
- detoxification of tissue
- improves tissue quality
- reduces limb circumference

The clinical effects of deep oscillation are documented in over 60 studies. You can find more information here and read the individual studies.

Just scan the QR code with your Smartphone.





DEEP OSCILLATION® Personal Home – simple and straightforward

Use of DEEP OSCILLATION® Personal Home is **simple and safe**. You hold a titanium contact element between the fingers of one hand. With the other hand you circle the hand applicator over the affected tissue. Two different-sized attachments allow you optimum treatment of the affected areas.

Contact with the oscillator head transmits the vibrations of the deep oscillation up to 8 cm deep into the tissue. Electrostatic pulses loosen up the tissue via the effect of friction and attraction. Indurations are loosened and fluids can be better transported away.

One application lasts between 5 and 12 minutes, depending on the severity of the disease and the phase of treatment chosen.

This type of treatment is extremely soft and gentle. The vibrations that occur provide patients with a pleasant sensation.



You can happily use DEEP OSCILLATION® Personal Home **wherever** you want. The device is small and easy to carry with you. You can even take it on holiday.

Effective relief of your complaints is just a flick of the wrist and a few minutes away. Visible and tangible results are usually immediate.

Assembly and operation of the device are easily comprehensible. Thanks to the intuitive single-button operation you simply choose the desired program from the **preset treatment** programs. You can immediately start the desired application.

Use deep oscillation whenever you need to.



DEEP OSCILLATION® Personal Home as a complement to other treatments

Compression stockings are included in the recommended measures for lipedemas. They reinforce the effect of a treatment by deep oscillation with a firm hold for the tissue.

Manual Lymphatic Drainage (MLD) has a proven positive effect on symptoms in the area of lipedemas. The treatment is however very time-consuming. It is not unusual for two applications per week to be required. Self-treatment with Deep Oscillation can be used to increase the intervals between therapy units. This gives you more flexible in your time management.




DEEP OSCILLATION® Personal Home

- accompanies you at each stage of the disease
- is available for you anywhere and at any time
- helps in acute cases with pressure pain and discomfort
- provides for gentle and at the same time very efficient relief of the pains
- complements treatment by lymph therapists
- offers users a significant improvement in their quality of life

DEEP OSCILLATION® Personal Home is an approved medical product.

Patients testify to their positive experiences with DEEP OSCILLATION® Personal Home.

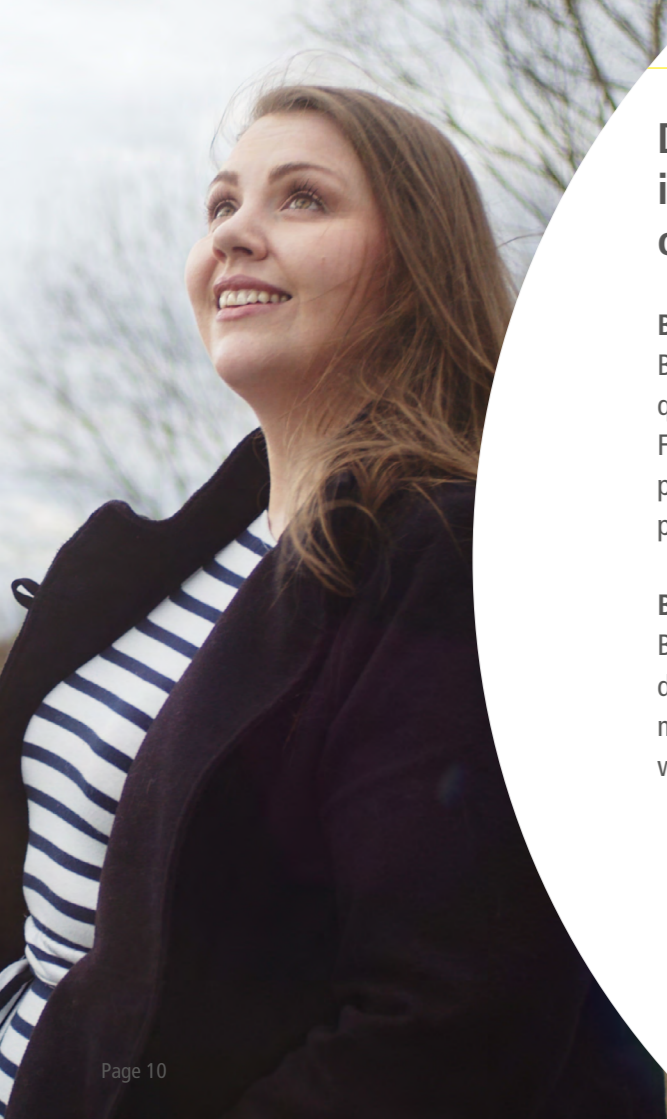

Deep oscillation is also recommended in particular for the pre- and post-treatment of liposuction

Benefits of preoperative application

By loosening and softening the tissue, during the intervention an increased quantity of body fat can be removed. This leads to visually better results. Furthermore this leads to less bleeding and bruising, meaning that fewer postoperative complaints occur and accelerated regeneration can take place.

Benefits of postoperative application

By being capable of early use after the operation, Deep oscillation demonstrably leads to reduction in pain, has the effect of promoting movement and reducing edemas, and promotes the process of healing the wound.

DEEP OSCILLATION® Personal Home

- Generous color display
- Intuitive user guidance with the PHYSIOMED single-button operation
- With preprogrammed treatments from the most important areas of application



DEEP OSCILLATION® Personal Home for your treatment at home

Are you interested in DEEP OSCILLATION® Personal Home?

Test Deep Oscillation without obligation at home with
our attractive financing offer.

Get in touch!

Contact us directly online.

Just scan the QR code with your Smartphone.



PHYSIOMED ELEKTROMEDIZIN AG
Hutweide 10
91220 Schnaittach
Germany

Phone +49 9126 2587-0
Fax +49 9126 2587-25
e-mail info@physiomed.de
Web www.physiomed.de