

# **DEEP OSCILLATION®**

Unique, non-invasive, atraumatic therapy technique with significant pain and oedema relieving effects






### DEEP OSCILLATION®

DEEP OSCILLATION<sup>®</sup> is a unique international patented non-invasive, non-traumatic therapy method. Its special structure allows you to create biologically effective oscillations in the treated tissue using electrostatic attraction and friction. In contrast to other therapies, these pleasant oscillations have a gentle and deep-acting effect on all tissue components (skin, connective tissue, subcutaneous fat, muscles, blood and lymph vessels).

For the treatment, the patient holds a titanium neutral element loosely between the fingers. The pleasant therapy effect of deep oscillation is created beneath special gloves of the therapist or a specially designed hand applicator (second contact) circling over the tissue.

The exceptionally gentle action, early implementation and specific clinical effects make DEEP OSCILLATION<sup>®</sup> a unique therapy option, which is being used increasingly in medical fields that are not typically focused on by conventional physical therapies.

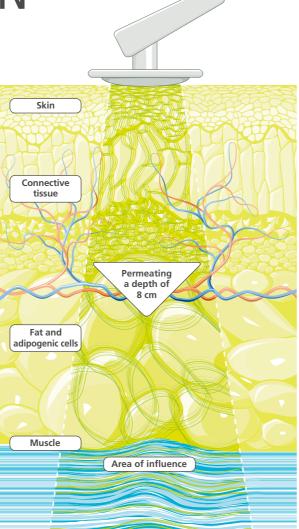



The following effects of DEEP OSCILLATION® are clinically proven:

- High pain-reducing potential (both for traumatic and chronic pain syndromes)
- Prevention and reduction of secondary and primary lymphoedema
- Muscular relaxation, movement-promoting effect, mobilisation
- Promotes wound healing, particularly for burns





## **Fields of application**

#### Pre- and postoperative therapy

With deep oscillation, oedemas can be relieved pre- and postoperatively. Because it can be used at an extremely early stage, wound healing is stimulated and accelerated, local inflammation processes are inhibited and pain is significantly reduced over a sustained period. Deep oscillation is therefore frequently used as a routine adjuvant post-operative therapy, for instance in oncology, neurology and traumatology.

#### Lymphoedema

Use of deep oscillation reduces volume in the case of primary and secondary lymphoedema. The therapy is also increasingly used with success for lipoedema. Skin conditions and lymphatic drainage are improved.

#### Mechanical traumata and damage from overstraining

In the case of traumata and damage from overstraining, deep oscillation has a direct oedema- and pain-reducing effect and consecutively stimulates self-mobilisation in pain-relieved areas, thus enabling an early return to active life. For sportspersons it means an earlier commencement of active forms of therapy and training, for 'normal' patients a swifter return to activities in daily life.

#### Training aftercare and performance stabilisation in sport

In training aftercare, deep oscillation is known for its rapid, muscle-relaxing and pain-relieving effects, and for its direct effect against microtrauma (muscle aches). Roughage and cell remains are removed more guickly through the treatment. This promotes a more effective nutritional supply to the muscle cells, thus accelerating the restoration to optimal performance. Regeneration times in the systematic training process can thus be reduced.

#### Deep oscillation is recommended by:

www.wittlinger-therapiezentrum.com Center for excellence in lymphedema treatment Lymphedema rehabilitation center – Treatmen Academy – Training – Research

In indications such as fibromyalgia syndrome, Sudeck's dystrophy etc., which are accompanied by chronic pain, deep oscillation has a strong pain relieving effect, which often allows for a reduction of medicine intake. The treatment promotes mobilization to a great extent, alleviates muscle stiffness and tackles impairments in activities of daily life, with consequent positive effect on fatigue, fears and depression.

In brain stoke rehabilitation deep oscillation is used for semi-paralysed regions of the body to improve trophicity and reduce spasms in affected areas, and also for lymphatic drainage, for instance to treat lymphoedema in the head area.

#### Burns

In the case of second-degree burns wound healing is significantly accelerated and qualitatively improved by deep oscillation.

#### Chronic pain syndromes


#### Neurortehabilitation



## DEEP OSCILLATION® Evident

Two-channel deep oscillation unit for professional use



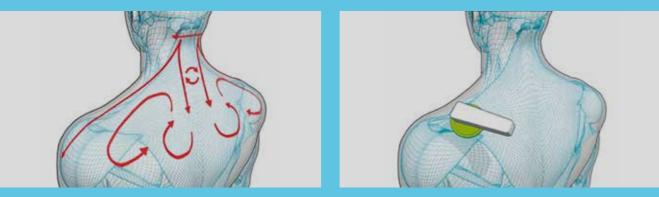

xtensive therapy and dosage suggestions and detailed animations illustrating treatment, which can be viewed during therapy at the touch of a button

DEEP OSCILLATION<sup>®</sup> Evident allows you to begin treatment with maximum speed and ease: direct selection of deep oscillation parameters, over the indication index or program memory. During treatment you have a constant overview of all values and timers. With decades of proven use, the PHYSIOMED one-button operation allows fast intuitive operation in combination with the touch screen.

Deep oscillation characteristics can be adjusted precisely to the desired treatment over the individual setting of frequency (or frequency band), duty cycle, burst function and therapy duration. The extensive indication list with practical information, graphics, animations and patient database with 'potpourri' function additionally ease use.

### Up to three indication menus can be selected for DEEP OSCILLATION<sup>®</sup> Evident:

- $\cdot$  CLINICS (for use in clinics, hospitals and physiotherapy practices)
- SPORTS (for use in sport medicine centres and clinics)
   AESTHETICS (for use in aesthetic medicine)



Swivel and tilt monitor mount

Therapy cards for treatment proposals

Technical Data	
Protection class	1, type BF
Power connection	$100 - 240 \text{ VAC} \pm 10\%$
Mains frequency	50 / 60 Hz
Current consumption	0.1 A (at 230 V) or 0.2 A (at 115 V)
Power consumption	20 – 30 VA
Output voltage max.	400 V
Load impedance	10 MΩ
Output frequency	5 – 250 Hz
Dimensions (W x H x D)	260 x 350 x 370 mm (monitor unfolded)
Weight	6.2 kg

#### Special features DEEP OSCILLATION®

Two-channel deep oscillation

Frequeny range from 5 – 250 Hz

Individual setting of frequency, frequency bands, duty cycle, duration of therapy and burst function

Contact elements made from biocompatible titanium

Special hand applicators in different sizes for all-over and local treatment Easily interchangeable treatment membranes guarantee

a hygienic application

Individual programmes as well as suggested treatments from the indication index can be easily stored on the DEEP OSCILLATION® therapy card and used with DEEP OSCILLATION® Personal

#### **General features**

Perfected user guidance through combination of touch screen and PHYSIOMED onebutton operation

8" colour monitor

Swivel and tilt mount for optimum monitor alignment at all times

Comprehensive overview of the therapy parameters including all therapy timers

Fastest therapy start: direct, through program memory or indications index

Treatment index with intelligent filtering functions based on body region or per alphabet for quick location of the desired treatment proposal

Extensive therapy and dosage suggestions and detailed animations illustrating treatment, which can be viewed during therapy at the touch of a button

Patient database for max. 100 entries: up to 5 treatment levels can be saved and configured as potpourris per patient

Multifunctional intensity controls with emergency stop function and quick switching between channels



Special hand applicators in various sizes

#### Standard accessories

Applicator handholds
Connection cables DEEP OSCILLATION®
Connection cable grey for adhesive electrodes
Mains cable
Operating instructions
Oscillator heads Ø 5 cm
Oscillator heads Ø 9.5 cm
Patient leads DEEP OSCILLATION®
PHYSIOPADS adhesive electrode for DEEP OSCILLATION® (set of 4)
Powder
Special gloves size M (100 pcs.)
Therapy cards DEEP OSCILLATION®
Titanium neutral elements
Trolley Evident

## DEEP OSCILLATION® Personal

DEEP OSCILLATION® unit for mobile use or to continue treatment at home



DEEP OSCILLATION® Personal can be used to read and retrieve program sequences previously written on the DEEP OSCILLATION® therapy card with the DEEP OSCIL-LATION® Evident unit in the clinic. This enables personalised mobile therapy on the ward and continuation of treatment in the patient's home according to the desired specifications. This way you can guarantee the sustainability of your treatments until the next appointment in your clinic or treatment room.

DEEP OSCILLATION® Personal is also available with the Basic and Pro therapy cards. The Basic version offers a wide selection of the most important indication settings with pre-programmed treatment parameters, which usually consist of different treatment sections of different frequencies. In addition, the Pro therapy card allows all parameters to be set manually as required for up to three therapy sequences of a treatment.





Therapy cards for treatment proposals

e-button operation

Tec	hni	cal	data	

reenneur uutu	
Protection class	II, type BF
Input voltage	
Power supply	
Output voltage max.	
Load impedance	
Output frequency	5 – 50 Hz

Mains frequency	50 – 60 Hz	



#### Special features DEEP OSCILLATION®

One-channel deep oscillation

Frequency range from 5-250 Hz

Contact elements made from biocompatible titanium

Special hand applicators in various sizes for both large or local treatments with easily replaceable treatment membranes for hygienic application (optional accessory)

#### **General features**

IIntuitive PHYSIOMED one-button operation

Colour monitor

Comprehensive overview of the therapy parameters

Fastest therapy start over the indication index\* or programmable therapy card DEEP OSCILLATION $^{\odot}$  therapy card

Extensive treatment index\* featuring therapeutic information, dosage proposals and application graphics

Battery-driven



ecial hand applicators in various sizes

#### Standard accessories Pro

Applicator handhold set, consisting of: 1 Hand applicator set 2 Oscillator-head Ø 5 cm 2 Oscillator-head Ø 9,5 cm 1 Connection cable DEEP OSCILLATION®
Charging device
Connection cable DEEP OSCILLATION®
Connection cable grey for adhesive electrodes
Operating instructions
PHYSIOPADS adhesive electrode for DEEP OSCILLATION $^{\odot}$ (set of 4)
Powder
Special gloves size M (100 pcs.)
Therapy card Pro
Titanium neutral element
Transportation bag
tandard accessories Basic
Applicator handhold
Charging device
Connection cable DEEP OSCILLATION®
connection cable DEEL OSCIELATION
Connection cable grey for adhesive electrodes
Connection cable grey for adhesive electrodes
Connection cable grey for adhesive electrodes Operating instructions
Connection cable grey for adhesive electrodes Operating instructions Oscillator-head Ø 5 cm
Connection cable grey for adhesive electrodes Operating instructions Oscillator-head Ø 5 cm PHYSIOPADS adhesive electrode for DEEP OSCILLATION® (set of 4)
Connection cable grey for adhesive electrodes Operating instructions Oscillator-head Ø 5 cm PHYSIOPADS adhesive electrode for DEEP OSCILLATION® (set of 4) Powder
Connection cable grey for adhesive electrodes Operating instructions Oscillator-head Ø 5 cm PHYSIOPADS adhesive electrode for DEEP OSCILLATION® (set of 4) Powder Special gloves size M (100 pcs.)

1 Transportation bag

### Clinical effects and studies

#### Pain reduction


Many medical studies show that deep oscillation reduces pain. The effect in the case of trauma-induced pain is seen in a reduction of noxious inflammatory potential (inflammation syndrome and, among other, Calor, tissue acidosis and prostaglandin) of TRPV1 pain receptors, explained by the mechanical distribution and intensified interstitial drainage via deep oscillation.

In an ex-vivo study Boisnic and Branchet (2013) found significantly fewer TRPV1-expressing keratinocytes than in the control group following deep oscillation treatment. The findings from ex-vivo skin models also confirm a statistically significant pain-reducing potential through deep oscillation.

#### Volume change

Several studies confirm the oedema-reducing effect of deep oscillation. Oscillation effects a 'stirring' of the basic substance, thus promoting the removal of interstitial fluid and substances (proteins, cell debris, neurotransmitters etc.). Interstitial septa and fissures are kept open by the mechanical activation, which helps interstitial drainage. This significantly reduces both local oedema and aseptic inflammation. Proof was also established of a significant reduction in swellings in the area of the wound, due to the treatment. In chronic conditions the treatment helps to disperse fibrosis and to diminish hardening of the tissue.


Ex-vivo tests show that deep oscillation also results in a significant reduction of dermal oedema compared to the control group. Significantly lower values than in the control group were also recorded in the histological evaluation of the percentage of dilated capillaries in the treatment group. Clear differences were likewise established in a histological planimetric measurement of the surface of dilated dermal capillaries (Boisnic and Branchet 2013). The findings suggest a vasomodulating effect (moderate vasoconstriction) of capillaries as a mechanism with an anti-oedematous effect prompted by deep oscillation.

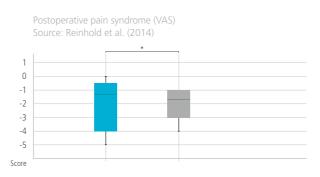

Median difference between day 7 and day 2 post OP




Source: Jahr et al. (2008)

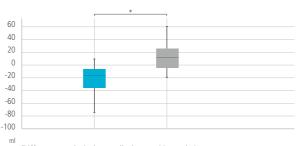

Difference admission – discharge (4 weeks)

Deep Oscillation n=11 Control n=10 \* p-value (Whitney U) = 0,007 (group comparison)

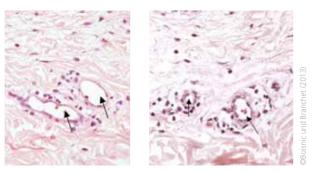

Analysis of vasodilation following HE staining (x 400): untreated skin model (on left). Vasoconstriction of dermal capillaries following two treatments with DEEP OSCILLATION® (on right).

#### Anti-flammatoy effect

Sportverl Sportschaden 23 1-4

PR06

23(1), 59-63.

102(4) 277-88

medicina estetica 30(4), 473-478.

po=2&idblog=110&p=1&n=dee

45-55

kurort2015430-35

Bioelectromagnetics Society. Kanazawa, 2.


Korkina L., Reinhold J., Rota L., Primavera G., Raskovic D. (2007): Treatment of Gynoid

Lipodystrophy (Cellulite) with Deep Oscillation®: A Pilot Clinical Study. 29th Annual Meeting of The

Deep oscillation checks acute and chronic inflammation by confining inflammation-promoting cell movement to the affected area. This is achieved by reducing the production of inflammation mediators and minimising water and protein loss from blood and lymph vessels. The findings of a study (ex-vivo) by Boisnic and Branchet (2013) confirm these clinical experiences: DEEP OSCILLA-TION<sup>®</sup> achieves a significant reduction in the prominent pro-inflammatory cytokine IL-8 compared to the control group.

#### The physiological effectiveness of deep oscillation has been documented in numerous publications based on experimental and clinical studies:

Aliyev R., Mikus E.W.S., Reinhold J.G. (2008): Hochsignifikante Therapieerfolge mit DEEP OSCILLA-TION<sup>®</sup> in der orthopädischen Rehabilitation. Orthopädische Praxis 44, 448-453. Aliyev R.M., Reinhold J., Seidov I.I., Mikus E.W.J. (2009): Ergebnisse mit DEEP OSCILLATION® bei Verletzungen des oberen Sprunggelenkes. Orthopädische Praxis 45, 502-506. Alivev R. (2009): Klinische Wirksamkeit des Therapieverfahrens Tiefenoszillation bei Sportver Aliyev R.M. (2012): Better Functional Results of Conservative Treatment in Fresh Lateral Ligament Injuries of the Ankle with Additional Deep Oscillation. Phys Med Rehab Kuror 22, 9–15. Andreev A. (2015): Deep oscillation, kinesiotaping and exercises in amateur football players after ACL reconstruction. XXIV International Conference on Sports Rehabilitation and Traumatology. London, Boisnic S., Branchet M.C. (2013): Anti-inflammatory and draining effect of the Deep Oscillation device tested clinically and on a model of human skin maintained in survival condition. Eur J Dermato Bolaños F. (2009): Efectividad de las oscilaciones profundas Hivamat personal en la patalogía artrósica de rodilla: Estudio de caso en adultos mayores. Universidad Santa Paula. Fistetto G., Iannitti T., Capone S., Torricelli F., Palmieri B. (2011): Deep Oscillation®: esperier terapeutico-riabilitative con un nuovo innovative strumento ad azione elettrostatica. Minerva Med Gao Y.-C., Peng C.-C., Peng R.Y. (2015): A long term chronic fibrotic adhesion of elbow muscles alleviated by applying hivamat 200 deep oscillation therapy. International Journal of Multidisciplinary Research and Development 2(1), 286-289. Gasbarro V., Bartoletti R., Tsolaki E., Sileno S., Agnati M., Coen M., Conti M., Bertaccini C. (2006): Ruolo dell'oscillazione profonda (Hivamat® 200) nel trattamento fisico del linfedema degli arti. La Gasbarro V., Bartoletti R., Tsolaki E., Sileno S., Agnati M., Conti M., Bertaccini C. (2006): Role of Hivamat<sup>®</sup> (deep oscillation) in the treatment for the lymphedema of the limbs. EJLRP 16(48), 13-15. Hernández Tápanes S., Suárez A., Bravo Acosta T., Wilson Rojas R., Fernández Prieto B., Cabrera Morales M. (2009): Valor de la terapia con oscilaciones profundas en la cicatrización de las quemaduras AB. Rev Cub MFR 2(1) [revista en la Internet]. Cited 05.02.2016; available from www.sld. cu/verpost.php?pagina=1&blog=http://articulos.sld.cu/revrehabilitacion/&post\_id=171&c=3734&ti-Hernández Tápanes S., Socas Fernández M., Iturralde Y., Addiel Suáres Fernández A. (2018): The Effect of Deep Oscillation Therapy in Fibrocystic Breast Disease, A Randomized Controlled Clinical Trial, International Archives of Medicine Vol. 11 No. 14. doi: 10.3823/2555. Hinman M.R., Lundy R., Perry E., Robbins K., Viertel L. (2013): Comparative Effect of Ultrasound and DEEP OSCILLATION® on the Extensibility of Hamstring Muscles. Journal of Athletics Medicine 1(1), Horn A., Mischler B. (2015): Kurzzeitwirkung von Tiefenoszillation auf die Fein- und Grob-motorik bei Multiple-Sklerose-Patienten und -Patientinnen mit Funktionseinschränkungen der Hand: Eine Einzelfallstudie. Berner Fachhochschule Fachbereich Gesundheit. Ivanova D.A., Khan M.A., Lyan N.A., Mikitchenko N.A. (2015): The application of the pulsed low-frequency electrostatic field for the combined treatment of the children presenting with bronchi asthma. Voprosi Kurortologi, Fosioterapi i Letschebniy Fisitscheskoi Kulturiy 4 (30-35). doi: 10.17116/ Jahr S., Schoppe B., Reisshauer A. (2008): Effect of treatment with low-intensity and extremely lowfrequency electrostatic fields (Deep Oscillation) on breast tissue and pain in patients with secondary breast lymphoedema. J Rehabil Med 40(8), 645-50. Johanning-Csik F. (1994): Behandlung postpartaler Brustschmerzen und -spannungen mit dem Intensiissystem Hivamat. Medical dissertation. Erlangen - Nürnberg. Khan M.A., Ivanova D.A., Ljan N.A., Lukina O.F. (2012): Application of the pulse low-frequency electrostatic field at bronchial asthma of children. Russian Journal of Rehabilitation Medicine 1, 21-32. Kashilska Y., Petkov A., Micheva P., Batashki A., Batashk Z. (2015): Improving the guality of life through effects of treatment with low intensity extremely low-frequency electrostatic field with DEEP OSCILLATION® in patients with breast cancer with secondary limfadem to patients treated with standard lymph equipment. Medicine V(1), 381-387.

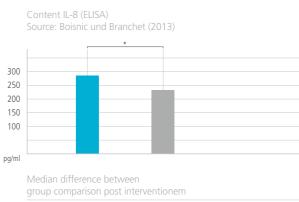

- Deep Oscillation n=4 Control n=4
- \* p-value (student) = 0,005
- Kraft K., Kanter S., Janik, H. (2013): Safety and effectiveness of vibration massage by deep oscillations: a prospective observational study. Evid Based Complement Alternat Med.;2013:679248. doi: 10.1155/2013/679248. Epub 2013 Oct 3.
- Mikhalchik E., Titkova S., Anurov M., Suprun M., Ivanova A., Trakhtman I., Reinhold J. (2005): Effects on blood parameters of Deep Oscillation. 1st International Conference on Skin and Envir Moscow-St. Petersburg, 59.
- Mikhalchik E., Titkova S., Anurov M., Suprun M., Ivanova A., Trakhtman I., Reinhold J. (2005): Wound Healing Effects of Deep Oscillation. 1st International Conference on Skin and Environment. Moscov St. Petersburg, 71.
- Nourollahi S., Mondry T.E., Herbst K.L. (2013): Bucher's Broom and Selenium Improve Lipedema: A Retrospective Case Study. Altern Integ Med 2(4), 119.
- O'Brien C.P., Watson A. (2016): Deep Oscillation® Therapy in the Treatment of Lateral Epicondyalgia: A Pilot Randomized Control Trial. J Sports Med Doping Stud 6(3), dx.doi.org/10.4172/2161-
- Reinhold J., Deeva I., Korkina L., Schaper K., Krummenauer F. (2014): Randomisierte Pilotstudie zur Quantifizierung des patientenseitigen Nutzens der Beeinflussung primärer Wundheilungsprozesse durch Tiefenoszillation. Z Orthop Unfall 152(3), 260-264.
- Sänger H.-F. (1995): Der Einfluß von HIVAMAT® 200 (histologisch-variable Technik) auf die Entstehung der radiogenen Fibrose bei brusterhaltend therapierten Patientinnen mit Mammakarzinom. Medical dissertation, Erlangen - Nürnber
- Scannavini P., Bitocchi M., Rossi M., Girvasi L. (2012): Lesioni muscolari da sport: percorsi di riatletizzazione. Scienze motorie, ortopediche, riabilitative 60, 31-35.
- Schönfelder G., Berg D. (1991): Nebenwirkungen nach bruster
- Mammakarzinoms. Erste Ergebnisse mit Hivamat. gynäkol. prax. 15, 109-122.
- Sporbeck B., Mathiske-Schmidt K., Jahr S., Huscher D., Becker M., Riemekasten G., Taufmann I., Burmester G.R., Pögel S., Reisshauer A. (2011): Effect of biofeedback and DEEP OSCILLATION® on Ravnaud's phenomenon secondary to systemic sclerosis: results of a
- controlled prospective randomized clinical trial. Rheumatol Int 32(5), 1469-73.
- von Stengel S., Teschler M., Weissenfels A., Willert S., Kemmler W. (2018): Effect of Deep Oscillation as a Recovery Method after Fatiguing Soccer Training: A Randomized Cross-Over Study, Journal of Exercise Science and Fitness, doi:10.1016/j.jesf.2018.10.004.
- Teo I., Coulborn A., Munnock D.A. (2016): Use of the HIVAMAT<sup>®</sup> 200 with manual lymphatic drainage in the management of lower-limb lymphoedema and lipoedema. Journal of Lymphoedema 11(1), 49-53
- Theys S., Deltombe T., Legrand C., Hanson P. (2008): Manual Drainage with or without DEEP OSCILLA-TION® in Lower Extremity Oedema. J Rehabil Med Suppl 47, 62.
- Trybulski R. (2008): Wykorzystanie cystemu Hivamat 200 w leczeniu ran. Rehabilitacja w Praktyce 1,
- Trybulski R. (2016): Fizykalne metody drenažu limfatycznego we wczesnej fazie fizjoterapii pourazowe narzadu ruchu. 27-42.
- Trybulski R. (2016): Metoda DEEP OSCILLATION w leczeniu zespołu stopy cukrzycowej. PRAKTYCZNA fizioterapia & rehabilitacia 68, 22-29.
- ybulski R., Zebrowska A. (2016): Obrzek Limfatyczny/Lymphatic Oedema. Miedzynarodowa Konferencja Sekcji Limfologicznej Polskiego Towarzystwa Flebologicznego 22-23. Scientific Poster at Interna nal Congress of the Lymphology Section of Polish Society of Phlebology. Wrocław, April 22-23.
- Trybulski R., Zebrowska A., Marcol W., Roczniok R., Kepa K., Kilianski M. (2016): Wykorzystanie głebokiej oscylacji i elektrostymulacji miesni gładkich w niwelowaniu wybranych parametrów zmecze nia miesniowego. Fizjoterapia Polska 2(16), 14-30.
- Winkelmann Z.K., Roberts E.J., Games K.E. (2018): Acute Effects and Perceptions of Deep Oscillation herapy for Improving Hamstring Flexibility. Journal of sport rehabilitation 2018 Jun 25:1-7. doi: 10.1123/jsr.2017-0044.
- Yashkov A.V., Gazdieva E.M., Badyanova I.S. (2007): Efficacy of intermittent low-frequency electrostatic field in the sanatorium-based complex treatment of patients with chronic obstructive pulmonary disease. Kurortnive Vedmosti 3(42), 62-63.
- Zebrowska A., Trybulski R., Roczniok R., Marcol W. (2017): Effect of Physical Methods of Lymphatic Drainage on Postexercise Recovery of Mixed Martial Arts Athletes. Clin J Sport Med 2017;0:1-8. doi: 10 1097/ISM 000000000000485
- Zehtindjieva M.G., Ioshinov B.R., Andonov D.R., Ilkov V.S., Bayraktarova A. (2013): Dee Oscillation – A Modern Additional Physical Modality For Analgesia In Patients With Back Pain. PRAEMEDICUS 29 85-90

Address PHYSIOMED ELEKTROMEDIZIN AG Hutweide 10 91220 Schnaittach Germany **Phone** +49 9126 2587 - 0

+49 9126 2587 - 25

Fax

Email info@physiomed.de

Web www.physiomed.de



